ManukaLind Patch de

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ManukaLind Patch en

Wound dressing. Made with Manuka honey (MGO 514+).

ManukaLind is a medical product for wound care and can be used in all stages of wound healing.

Instructions for Use

Read all the instructions in this leaflet carefully. It contains important information. Keep the Instructions for Use.

Areas of application / Indications

ManukaLind can be used for skin irritations and wounds. These include: Superficial wounds, skin irritations, cut wounds, abrasions, scalds, burns (1st and 2nd degree), acute and chronic wounds, ulcers, infected wounds, surgical wounds, operation wounds, necrotic wounds, diabetic wounds, covered wounds, decubitus ulcers, poorly healing wounds, bite wounds, scar care (for sutured wounds), foot and leg ulcers (venous, arterial)

The natural triple effect of ManukaLind – Cleansing, protection and wound healing

Thanks to its high osmolarity, ManukaLind has an antibacte-rial effect. Thus provides effective, natural protection against bacteria and invading germs.

Dressing changes are less painful and less traumatizing as the ointment reduces dressing adhesion due to ongoing moisture.

ManukaLind creates an ideal moist and optimum pH wound environment.

ManukaLind promotes wound granulation and counteracts wound odour.

Instructions for use

- Carefully clean the wound edges if necessary.
- Remove the patch from the packaging and place it on the wound. May be cut with sterile scissors if necessary.
- Apply ManukaLind generously. There must be full contact between ManukaLind and the wound bed.
- A suitable sterile dressing can be used to cover the wound.
- If interactive dressings are used (e.g. polyfoam or hydrocolloids), it must be ensured that they do not contain active ingredients, since no experience has been gained yet regarding the compatibility with ManukaLind.
- The dressing should be changed if it is saturated with exudate, or if the covering dressing leaks wound secretions.
- When removing / changing the dressing, if necessary, rinse / clean the wound with a saline solution and reapply ManukaLind.
- The currently valid guidelines for hygiene and infection prevention must be observed when caring for wounds or changing dressings. This includes at least thorough hand disinfection, the use of sterile disposable gloves and mouth/ nose protection.

Contraindications

- Not to be used with known allergy to bee products (e.g. honey, pollen, propolis etc.), thyme or lavender oil.
- Do not use on heavily weeping wounds, third degree burns, in deep tight cavities/wound areas or sinuses.
- Not suitable for children under 12 Months. Pregnant women and breastfeeding mothers after consulting with their doctor.
- There are no reports of elevated blood sugar levels with diabetes patients after use of the ointment, as the product is not reabsorbed. Nevertheless, diabetics should consult their doctor before using ManukaLind.

Safety Precautions and Warnings

- Only use one patch per patient. Do not reuse used patches! Use patches must be immediately disposed of after use.
- In case of contact with eyes, rinse thoroughly.
- After application, light burning or pulling sensation may occur. In case of ongoing discomfort remove dressing, rinse the wound with a saline solution and if necessary, consult your doctor.
- If allergic reactions occur, stop treatment immediately and contact your doctor.
- If the desired effect is not achieved within 3 weeks, discon-tinue use and consult your doctor.

Mandatory reporting

Please report all serious incidents related to ManukaLind to the manufacturer and the competent authority in your country.

Ingredients

Polyethylene patch, coated with ManukaLind ointment, consisting of macrogol, Manuka honey (MGO 514+), essential oils (lavender oil, thyme oil).

Disposal

Empty packaging can be disposed of with household waste.

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Byox Healthcare GmbH